

Maximum pH Plus D.S.

Double Strength Alkaline Hospital Instrument Cleaner & Pre-Soak Concentrate for Use in Washer Disinfectors

Product Properties:

- Alkaline detergent for use in washer disinfectors.
- Also suitable for use in ultrasonic baths and flexible endoscope processing.
- Safe for use on soft metals such as aluminium and Anodised aluminium.
- Concentrated strength enables savings on usage, storage and transport costs.

Material Compatibility:

Maximum pH Plus D.S. is designed for the automated cleaning of surgical instruments.

*Stainless Steel *Aluminium *Anodised Aluminium *Chromium Plated Materials *Synthetic Materials *Rubber *Latex *Glass *Ceramics *Copper *Brass *Metal Alloys

When used correctly in the correct dilutions, no material damage is to be expected.

Areas of Application:

- For use in washer disinfectors.
- Can also be used in ultrasonic baths.

Composition:

Maximum pH Plus D.S. contains amongst other ingredients:
Disodium Metasilicate CAS No 6834-92-0
pH 12-14 typical

Directions of Use:

When changing containers, wear rubber gloves, goggles and plastic apron. Place feed tube into bottom of container and check automatic feed equipment is switched on. **Dilution for Use:** Inject at 2mls per litre of wash water. **Temperature Range:** Between 45-60°C. **Minimum Contact Time:** 2 Minutes. **Shelf Life:** 18 Months.

Ordering Details:

Product Code:	1078D	1080D	1079D	1081D	101078D
Pack Size:	2x5L	UN 2x5L	10L	20L	1000L

