

Auto-Active8

Super Concentrated Machine Enzymatic Detergent for Cleaning Reusable Medical Instruments and Devices

Product Properties:

- Non-foaming.
- Non-scented.
- Multi-tiered enzymatic.
- Economical.
- Suitable for all makes of automated washers.
- Will promote the removal of bioburden from surgical instruments.



Material Compatibility:

Auto-Active8 is designed for the automated cleaning of surgical instruments.

*Stainless Steel *Chromium Plated Material *Synthetic Materials *Rubber
*Latex *Glass *Ceramics *Copper *Brass *Metal Alloys

When used correctly in the correct dilutions, no material damage is to be expected.

Areas of Application:

- For use in washer disinfectors.

Composition:

Auto-Active8 contains amongst other ingredients:
Monopropylene Glycol CAS No 57-55-6, Propan 2-OL CAS No 67-63-0, Subtilisin CAS No 9014-01-1
pH 7.5-8.7 typical

Directions of Use:

When changing containers, wear rubber gloves, goggles and plastic apron. Place feed tube into bottom of container and check automatic feed equipment is switched on. **Dilution Rate:** Inject 4ml per litre of wash water. **Temperature Range:** Between 27-60°C. **Minimum Contact Time:** 2 Minutes. **Shelf Life:** 24 Months.

Ordering Details:

Product Code:	1069	1054	1055	1054200
Pack Size:	6x500ml	2x5L	10L	200L