

Bravo T.W.2

Acidic Automated Trolley Washing Detergent

Product Properties:

- Alternative concentrated acidic detergent specifically designed to deal with the issue of soft anodised transport trolleys.
- Balances the wash water in decontamination process to reduce pH.
- Aqueous solution dispensed in usual way.
- Economical.



Material Compatibility:

Bravo T.W.2 is designed for the automated cleaning of trolleys and beds.

*Stainless Steel *Aluminium *Anodised Aluminium *Chromium Plated Materials *Synthetic Materials *Rubber *Latex *Glass *Ceramics *Copper *Brass *Metal Alloys

When used correctly in the correct dilutions, no material damage is to be expected.

Areas of Application:

- For use on instrument transportation trolleys.
- For use on hospital beds.

Composition:

Bravo T.W.2 contains amongst other ingredients:
Tetrasodium Ethylene Diamine Tetraacetate CAS No 64-02-8, Sodium Hydroxide CAS No 1310-73-2
pH 5-6 typical

Directions of Use:

When changing containers, wear rubber gloves, goggles, and plastic apron. Place feed tube into bottom of container and check automatic feed equipment is switched on. **Dilution Rate:** Inject 2-4ml per litre of wash water. **Temperature Range:** Between 45-60°C. **Minimum Contact Time:** 2 Minutes.

Ordering Details:

Product Code:	108910	1089	5089
Pack Size:	10L	20L	500L