

## Flexi-Clenz

### Neutral Manual and Machine Detergent for Cleaning Reusable Medical Instruments & Devices

#### Product Properties:

- Neutral detergent.
- Great cleaning power.
- Economical.
- Removes any organic traces e.g., blood, protein, fat, saliva, chemical residues found on and in flexible and rigid devices.
- Complies with BSG guidelines.
- Suitable for all makes and manufacturer of flexible and rigid endoscopes.

#### Material Compatibility:

Flexi-Clenz is designed for the manual cleaning of flexible and rigid endoscopes and reusable surgical instruments.

\*Stainless Steel \*Aluminium \*Anodised Aluminium \*Chromium Plated Material  
\*Synthetic Materials \*Rubber \*Latex \*Glass \*Ceramics \*Copper \*Brass \*Metal Alloys

When used correctly in the correct dilutions, no material damage is to be expected.



#### Areas of Application:

- For use with flexible and rigid endoscopes.

#### Composition:

Flexi-Clenz contains amongst other ingredients:  
Propan-2-OL CAS 67-63-0, Glycerine (Glycerol) CAS No 56-81-5  
pH 6-7.5 typical

#### Directions of Use:

Use to remove residues and deposits found on and in flexible and rigid devices. **Dilution for Use:** Inject at 2-4ml per litre of wash water. **Temperature Range:** Between 27-35°C. **Minimum Contact Time:** 2 Minutes. **Shelf Life:** 24 months.

**\*\*For machine use, follow manufacturers recommendations\*\***

#### Ordering Details:

|               |       |      |
|---------------|-------|------|
| Product Code: | 1036C | 1038 |
| Pack Size:    | 2x5L  | 20L  |