

# Neutra-Scope

## Neutral Manual and Machine Detergent for Cleaning Reusable Medical Instruments & Devices

### Product Properties:

- Neutral detergent.
- Great cleaning power.
- Economical.
- Removes any organic traces e.g., blood, protein, fat, saliva, chemical residues found on and in flexible and rigid devices.

### Material Compatibility:

Neutra-Scope is designed for the manual and machine cleaning of flexible and rigid endoscopes and reusable surgical instruments.

\*Stainless Steel \*Chromium Plated Material \*Synthetic Materials \*Rubber  
\*Latex \*Glass \*Ceramics \*Copper \*Brass \*Metal Alloys

When used correctly in the correct dilutions, no material damage is to be expected.



### Areas of Application:

- For use with manual and machine cleaning of reusable flexible and rigid endoscopes and reusable surgical instruments.

### Composition:

Neutra-Scope contains amongst other ingredients:  
Propan-2-OL CAS No 67-63-0, Monopropylene Glycol CAS No 57-55-6  
pH 7.5-8.7 typical

### Directions of Use:

Use to remove residues and deposits found on and in flexible and rigid devices. **Dilution Rate:** Inject 4ml per litre of wash water. **Temperature Range:** Between 27-35°C. **Minimum Contact Time:** 2 Minutes.

**\*\*For machine use, follow manufacturers recommendations\*\***

### Ordering Details:

Product Code:	1034	1034U	1035	103420
Pack Size:	2x5L	UN 2x5L	10L	20L