

Renovate Plus

Acidic Based Instrument Enhancing Detergent

Product Properties:

- Removes rust from stainless steel surgical instruments.
- Removes corrosion from stainless steel surgical instruments.
- Acidic based.

Material Compatibility:

Renovate Plus is designed for the cleaning, maintenance and removal of rust and corrosion from stainless steel surgical instrumentation.

Stainless Steel

When used correctly in the correct dilutions, no material damage is to be expected.

Areas of Application:

- For use in the manual washing phase only when cleaning instrumentation.
- For use with reusable surgical instruments.
- Can also be used in the cleaning/decontamination cycle to clean washer disinfector and internal tubes to prevent residual and water scales from sticking to the chamber walls. NO INSTRUMENTS IN THE WASH.

Composition:

Renovate Plus contains amongst other ingredients:

Phosphoric Acid CAS No 7664-38-2, Propan-2-OL CAS No 67-63-0, Isodecanol Ethoxylate CAS No 26183-52-8
pH 1-2 typical

Directions of Use:

Renovate Plus is for use in the manual wash phase only. **Dilution Rate:** Mix 5-10ml per litre hot water, depending on water and instrument condition. Put solution in a non-porous container. Place thoroughly cleaned and opened instruments into solution and leave to soak for 15 minutes. Heavily stained instruments soak for 30 minutes. Rinse thoroughly. **Temperature Range:** Between 30°C-35°C.

NOTE: Renovate Plus can be used in the cleaning/decontamination cycle to clean washer disinfector and internal tubes to prevent residual and water scales from sticking to the chamber walls. NO INSTRUMENTS IN THE WASH. **Temperature Range:** Between 30°C-80°C.

Ordering Details:

Product Code:	1063	1066	1065	1065U
Pack Size:	1L	12x1L	2x5L	UN 2x5L

