

Scope-Clenz

ScopeTech Flushing Solution

Product Properties:

- Flushing solution specifically formulated for use in the ScopeTech flushing device only.
- Low hazard solution that is safe in application for all members of staff.
- Odourless.
- Clear in solution.
- Supplied in the Ready to Use 300ml pot or 5L container.

Material Compatibility:

Scope-Clenz is designed for the daily flushing of ScopeTech devices only.

*Stainless Steel *Metal Alloys *Chromium Plated Material *Synthetic Materials
*Rubber *Latex *Glass *Ceramics *Copper *Brass *Metal Alloys

When used correctly in the correct dilutions, no material damage is to be expected.



Areas of Application:

- For use on the cleanse cycle of the ScopeTech unit.

Composition:

Scope-Clenz contains amongst other ingredients:
Alkyl (C12-16) Dimethyl Benzyl Ammonium Chloride CAS No 68424-85-1
pH 5.5-8 typical

Directions of Use:

Scope-Clenz is to be used on the Cleanse Cycle only, for use with the ScopeTech unit.
Connect directly to the flush pick-up connector and follow the appropriate instructions as detailed via the dispenser.

This product does NOT require diluting.

MUST BE THOROUGHLY FLUSHED BEFORE SCOPE PROCESSING OCCURS.

Ordering Details:

Product Code:	1002	1001
Pack Size:	30x300ml	2x5L